

**Follow these Instructions  
step-by-step:**

**LevMed Universal ECG Belt with Handles**




**Ensure the Patient Cable is connected  
to the ECG Device.**

## Connect the Banana\* Patient-Cable to the Belt:

1. Open the screws:



2. Insert the Banana connectors:

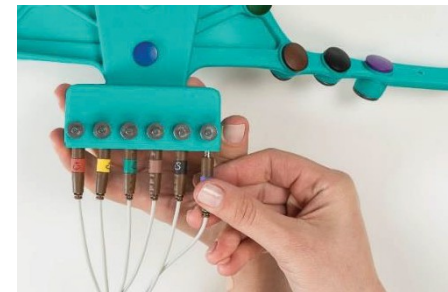

3. Close the screws:



\*For a Snap Patient-Cable click to the next page.

## Connect the Snap Patient-Cable to the Belt:

1. Close the screws tightly:




2. Connect the snaps:








Connect the Patient-Cable to each of the 4 clamps:


1. Open the screw.
2. Insert the banana connector / connect the snap.
3. Close the screw.

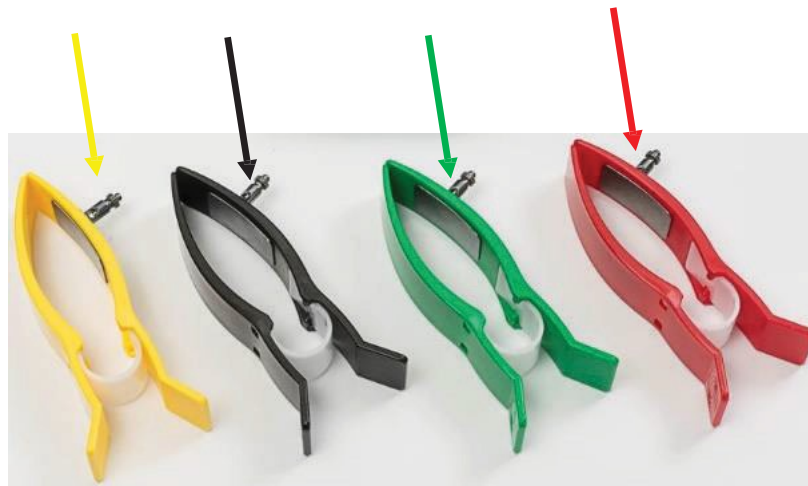

## Open or Remove the patient's shirt:

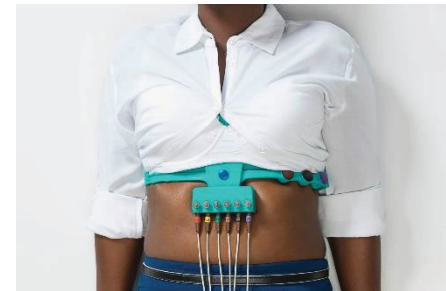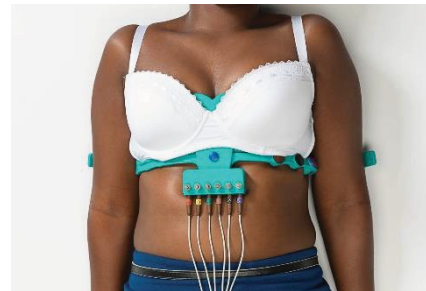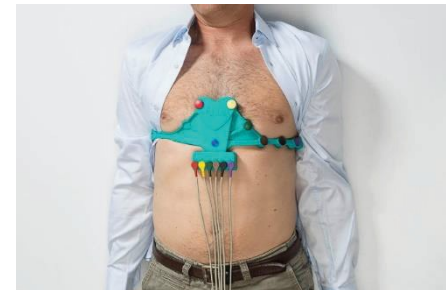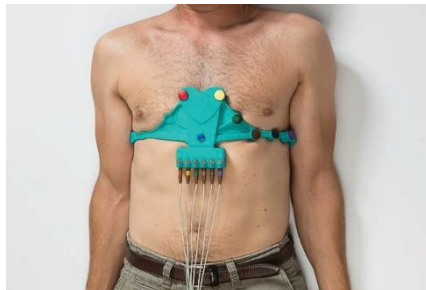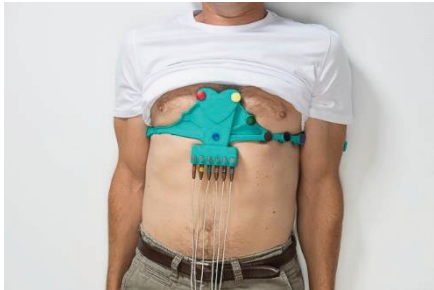

**Wet the 6 Electrodes with Tap water:**

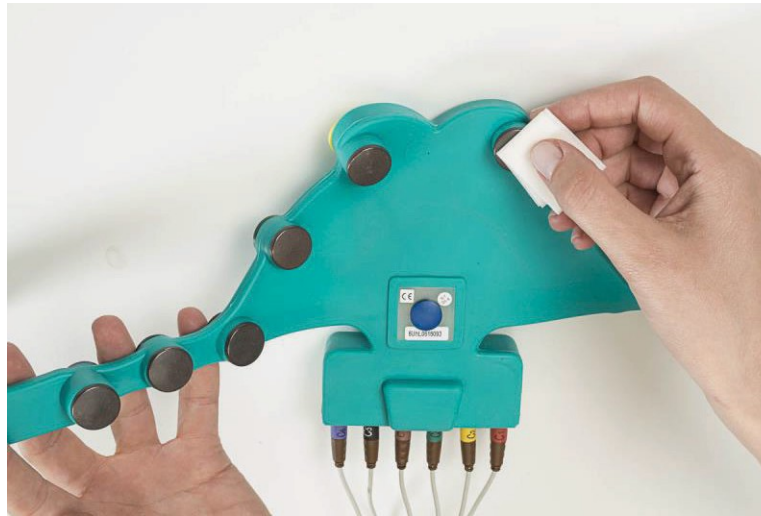

**Adjust the 2 handles to fit patient's arms:**







**If necessary, bend V1 & V2 to ensure electrode contact with patient chest:**





**Apply the belt on patient chest:**

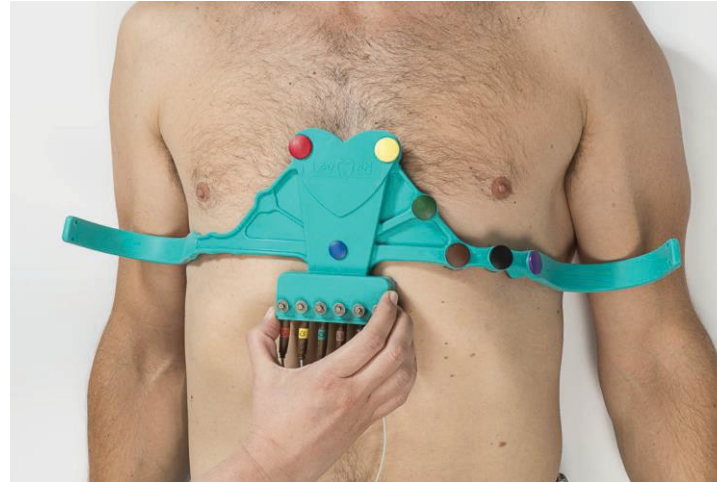

**Note: for women, apply the belt under the bra:**

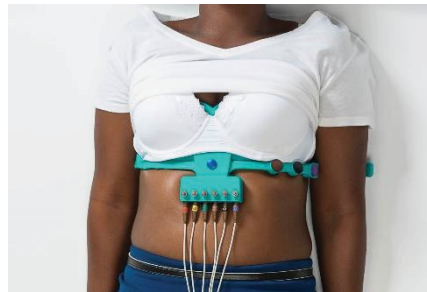


**Apply the handles on mid-arm,  
between elbow and shoulder:**

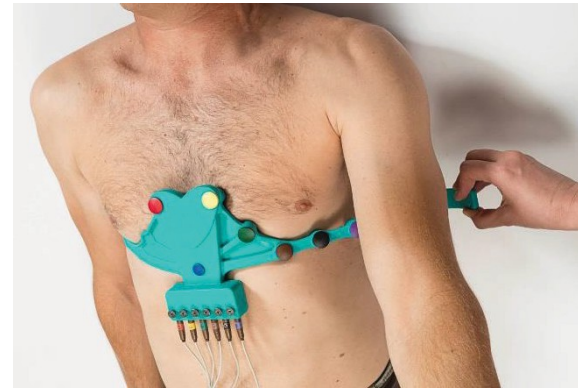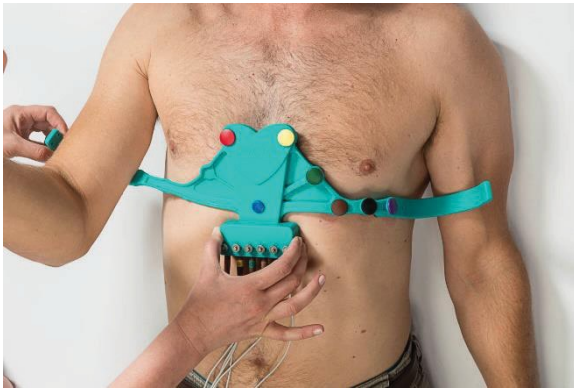

**Ensure all 6 electrodes are in contact  
with patient skin:**



**Wet the Electrode on each of the 4 Clamps  
with Tap water:**

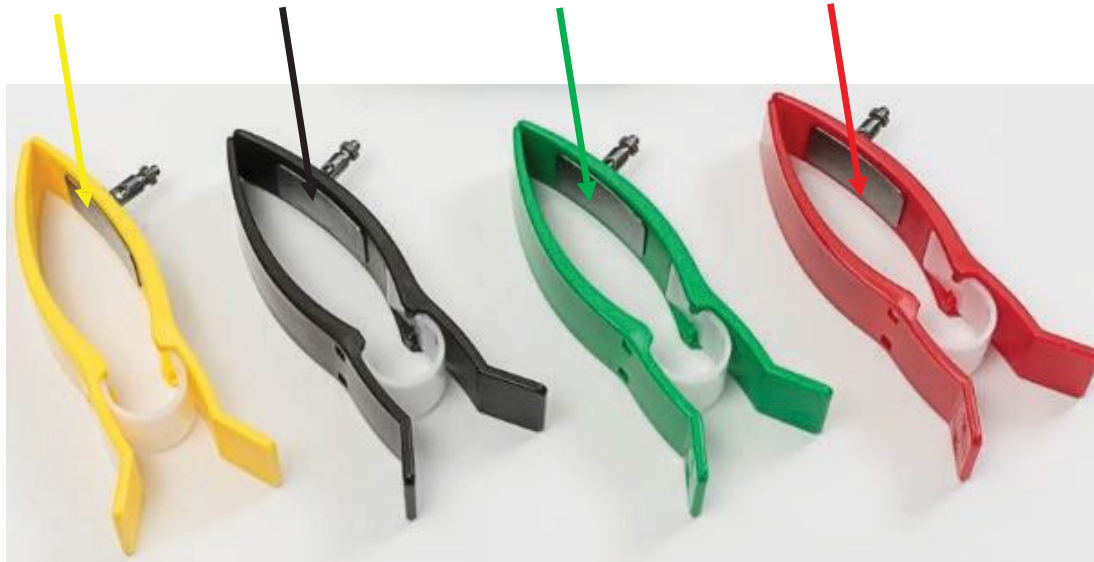


Apply the clamps on the limbs:

**Red = Right Arm**

**Yellow = Left Arm**


**Green = Left Leg**

**Black = Right Leg**

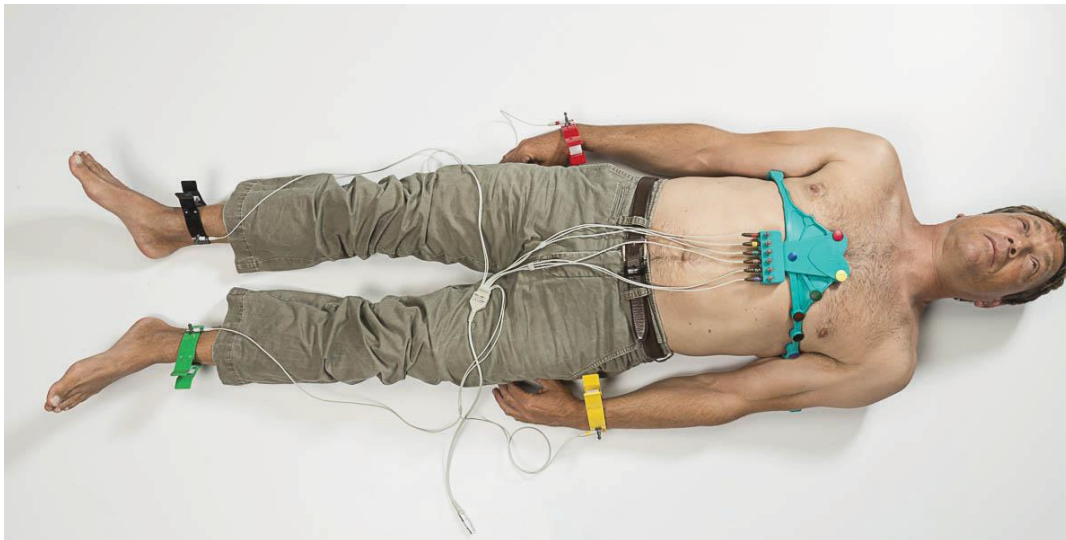


Ensure the patient is relaxed and does not strain his muscles.

Ask the patient not to move or talk during the assessment.

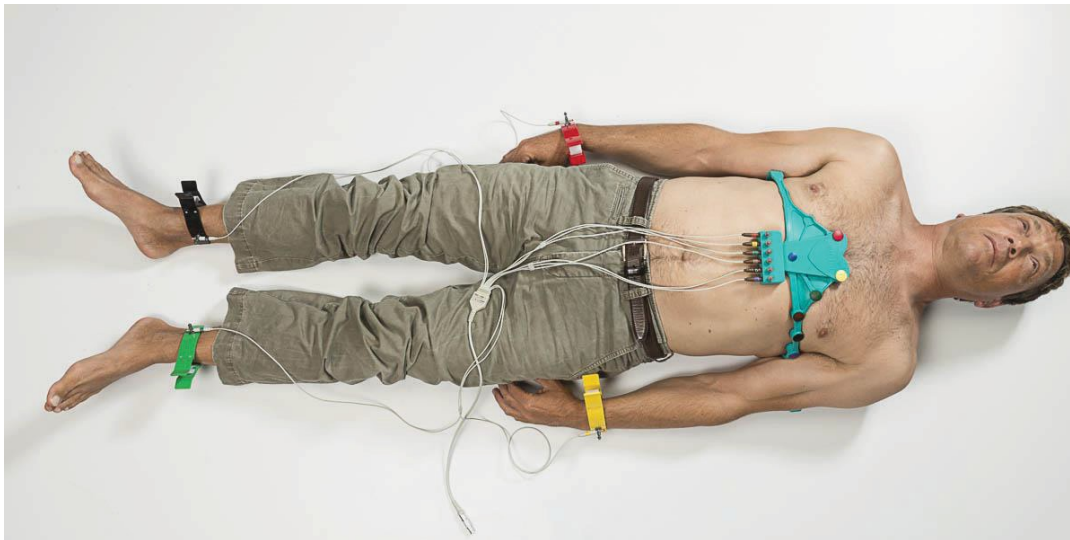

Press 'Start' on the ECG Device to take ECG Reading.

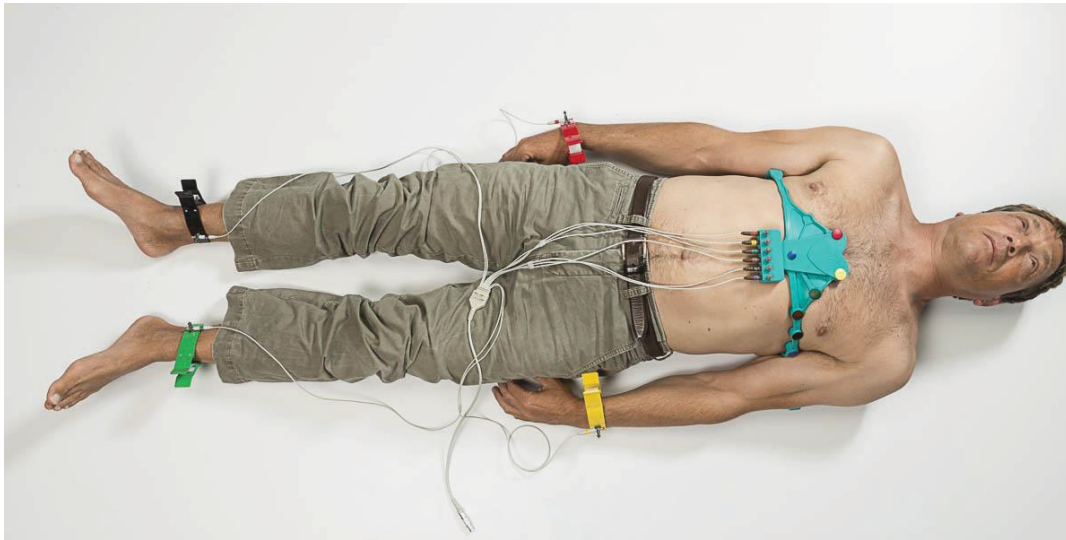





# VENTURA

Thank you for using  
LevMed ECG Belt!

*Marketed in India by:*

**VENTURA BUSINESS SOLUTIONS**

**BANGALORE**

302/2, 1st Floor, HAL II Stage 6th  
Main, 4th Cross, Indira Nagar  
Bangalore-560038, Karnataka, INDIA  
Mobile: +91 7829496000

**NEW DELHI**

109/13, 1st Floor, Amritpuri-B  
Opp. ISKCON Temple  
East of Kailash, New Delhi-110065, NCT, INDIA  
Mobile: +91 9910895444

**VENTURA**

[www.venturabiz.in](http://www.venturabiz.in)